

Click to verify































rollator andador déambulateur EU Authorized Representative Drive Medical LTD Ainley's Industrial Estate Elland, West Yorkshire, United Kingdom HX5 9JP www.drivemedical.comwww.drivemedical.com Do not aempt to adjust or operate Rollator without carefully reading all instructions in this manual. Orthodic Hand Grips Wide Comfortable Seat Hanging Bag for Functionality & Storage Height Adjustable Folding Hinge for Portability Loop Locks Safest System Available Backrest assembly instructions 1. Remove all contents from carton. 2. Open frame and place up-side-down on the floor. Aach legs without brakes to the front of the walker by depressing push pin and sliding leg into the receptacle. Secure in place using the tightening knob, threading the knob in from the outside. Repeat for second leg, making sure both legs are set to the same height. 3. Aach back legs (with brakes) using the same process, making sure the wheels are on the outside of the rollator and are at the same height as the front wheels. Affix brake cable to the black portion of the leg using the "S" shaped clamp provided. 4. Insert handle tube into frame and set to desired height. Secure in place using tightening knob, threading the knob in from the outside. Repeat for second handle, making sure both are set to the same height. 5. Aach padded backrest by depressing the black thumb buons and sliding into the black bracket. Note that back rest can be folded up when not in use. safety precautions 1. Please make sure the Rollator is fully opened, seat is down and the folding bar under seat is locked in the down position. 2. Be sure that both knobs are fully tightened and are at equal heights. Test Rollator for stability before applying full weight to hand grips. 3. Test the loop locks before operating the Rollator. When wheels are locked (see illustrations for locking and releasing locks below), lock bar on rear wheels should prevent wheels from turning. If the wheels turn while brakes are locked, please take unit back to dealer for adjustment. When locks are released, the wheels should turn freely. 4. Always lock loop locks before siing and do not use seat when unit is on an incline or uneven ground. 5. Replace tires when significantly worn or damaged. 6. Do not have anyone push you while you are seated on the Rollator. This is a walking aid only and is not to be used as a transportation device. loop lock operation Always engage and disengage wheel locks on both sides simultaneously. 1. To Slow Down or Come to a Temporary Stop — Pull up slowly on the loop locks with four fingers until unit slows down or comes to a temporary stop. 2. To Temporarily Lock Wheels in Position — Press down on both loop locks with palms until you hear a "click" that will tell you wheels are locked. 3. To Release Wheel Locks — Push up on both loop locks with palms of hand to original position. height adjustment The height of the rollator can be adjusted in two ways to accomodate many different users. lifetime limited warranty handle height adjustment 1. Remove tightening knob from handle. 2. Set handle to desired height. 3. Replace tightening knob into handle, making sure to insert knob from the outside. 4. Repeat for second handle, making sure both handles are set to the same height seing. leg height adjustment 1. Remove tightening knob from leg. 2. Depress push pin and slide leg to desired height. 3. Replace tightening knob into leg, making sure to insert knob from the outside. 4. Repeat for second, third and fourth legs, making sure all are set to the same height seing. Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser. This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement. If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer. © 2006 Medical Depot, Inc. All rights reserved. Drive is a trademark of Medical Depot, Inc. Port Washington N.Y. 11050 USA Made in China